

POLYGLYCAN[®]-SA

(Hyaluronic Acid + Chondroitin Sulfates C4 & C6 + N-acetyl-D-glucosamine Sterile Solution)

DATA SHEET



Polyglycan-SA is indicated for the replacement of lost or damaged synovial fluid in horses and dogs. Polyglycan-SA contains naturally occurring components of synovia that play a central role in maintaining the homeostatic environment of the joint.

BENEFITS

- ✓ Help replenish lost synovial fluid with naturally occurring elements
- ✓ Assist in normalization of synovial fluid viscosity
- ✓ May be used for post-surgical lavage of the joint cavity and surfaces
- ✓ Over a million doses administered by trusted veterinarians since 2006
- ✓ Patented 3-component formula which has demonstrated characteristics beyond any of these components used individually.

LIST No	UNIT PACKAGE	CASE SIZE
1POL009	6 x 2.5 mL vials	240



Rx Caution: This device is restricted to use by or on the order of a licensed veterinarian.

See reverse for Administration & Dosage

POLYGLYCAN[®]-SA (Hyaluronic Acid + Chondroitin Sulfates C4 & C6 + N-acetyl-D-glucosamine Sterile Solution)



Sterile Post-Surgical Lavage, Synovial Fluid Replacement, Viscosupplement

Veterinary Use Only

CAUTION

This device is restricted to use by or on the order of a licensed veterinarian.

DESCRIPTION

Polyglycan-SA is a patented formulation designed to replace lost or damaged synovial fluid. Polyglycan-SA contains naturally occurring components of synovia that play a central role in maintaining the homeostatic environment of the joint.

COMPOSITION

Highly viscous aqueous solution of defined fractions of purified Hyaluronic acid, Chondroitin sulfates C4 & C6 in a 10% solution of N-acetyl-D-glucosamine.

Each vial contains:

Hyaluronic acid sodium salt.....	12.5 mg
Sodium chondroitin sulfate.....	.250 mg
N-acetyl-D-glucosamine.....	.250 mg

PROPERTIES

Glycosaminoglycans are important components of all extracellular tissue structures including cartilage and synovial fluid. The active components in Polyglycan-SA exhibit viscoelastic and polyionic properties similar in nature to synovial fluid.

Introduce Polyglycan-SA into the synovial space as a post-surgical lavage, synovial fluid replacement or viscosupplement to assist in the normalization of the joint.

DIRECTIONS

Instill Polyglycan-SA into the synovial space to replenish lost synovial fluid. Volume of product will vary with size of joint / patient. Quantity to be determined by Veterinarian Surgeon. Discard any unused portion of vial contents.

STORAGE: Refrigerate whenever possible. DO NOT FREEZE. Maintain product at or below 65°F (18°C) during shipment and storage.

WARNING

Do not administer to animals that are to be slaughtered for use in food. Keep out of reach of children.

DISPOSAL OF UNUSED PRODUCT OR WASTE MATERIALS DERIVED FROM THE USE OF SUCH PRODUCTS

Any unused product or waste materials should be disposed of in accordance with local requirements.

Patent Nos.: 6,979,679 / 7,485,629 / 8,455,458

MANUFACTURED FOR:

Bimeda, Inc.
Le Sueur, MN 56058
www.bimeda.com

N.A. CORP. ADDRESS:

Bimeda, Inc.
One Tower Lane, Oakbrook Terrace, IL 60181

For further information and queries, please contact Bimeda at (888) 524-6332 or US-Info@Bimeda.com

TAKE TIME



OBSERVE LABEL
DIRECTIONS

www.BimedaUS.com